



YOUR SOURCE FOR ATTACHMENT  
AND REMOVAL OF EEG ELECTRODES

**MORE THAN COLLODION!**

## **MAVIDON HD AIR PUMP AUTOMATIC SHUT-OFF**



### **Instructions:**

- Attache air hose in the quick release adaptor.
- Plug in a 110 Watt, - grounded outlet.
- Press foot pedal and pump will automatically turn on.
- Release foot pedal to turn off pump.
- Use electrode applicator, supplied, to help dry collodion.

### **Part Number MD0025**

### **Specs:**

- Volts 115
- Amps 2.6
- CFM 1.5
- PSI 40

### **Warranty**

- 1 year warranty on motor from date of purchase.

Distributed by: Mavidon® Lake Worth, FL 800-654-0385 [www.mavidon.com](http://www.mavidon.com)

Made in Italy 